ORTHOKIT 2.0


EN•DE•NL

CONTENTS

ORTHOKIT 2.0 EN	3
ORTHOKIT 2.0 DE	12
ORTHOKIT 2.0 NL	22


ORTHOKIT 2.0 EN

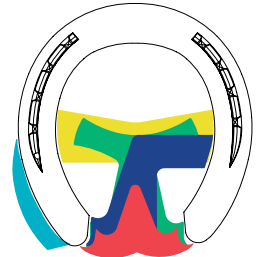• Z-Bar	4
• Center Bar	5
• Spider Plate	6
• Lateral Extension	7
• Heart Bar	8
Dimensions	9
Sizes	10
Comments / Disclaimer	11

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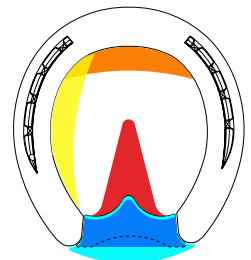ORTHOKIT 1.0

WERKMAN ORTHOKIT 2.0

In 2013 in cooperation with the university of Leipzig along with farriers and vets from around the world Werkman horseshoes developed the "ORTHOKIT 1.0".

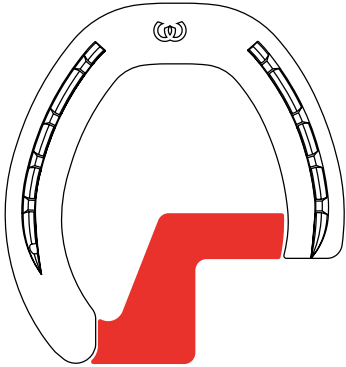
This innovative range of orthopaedic horseshoe inserts has been enthusiastically received by the farriery community around the world and as a result of their enthusiasm and feed back additional therapeutic inserts have been developed.

The "ORTHOKIT 2.0" remains true to the original ideas, enlarging the range of easy to use orthopaedic horseshoe inserts designed by the experience of those in the field and supported by scientific research from the University of Leipzig.

Werkman horseshoes thanks you for your passion and commitment in improving the standards of farriery and welcomes your thoughts and ideas for future development.

This brochure explains how the ORTHOKIT modifications are made and how they function.

Z-BAR 2.0

FIELD OF APPLICATION

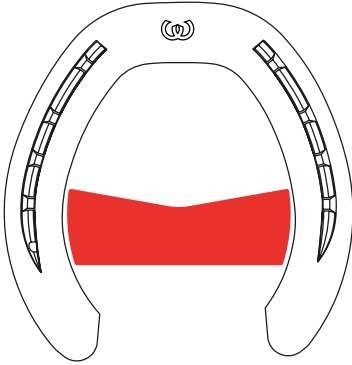
- Quarter cracks
- Sheared heels

EFFECT

- The Z-Bar enables you to float a complete heel quarter on solid ground. This can bring relief to the stressed / injured area.

CENTER BAR 2.0


FIELD OF APPLICATION

- Founder/Laminitis
- Pathologies of the deep flexor tendon
- Pathologies of the check ligament of the deep flexor tendon
- Chronic navicular disease
- Palmar foot pain
- Inflammations of the ligaments of the toe
- Hoof cracks

EFFECT

- The Central Bar has a wide range of applications. Using this bar in conjunction with a hoof packing can provide support to a specific area. Additionally, using the Central Bar without packing can provide protection.
- If you weld in the Central Bar in the middle of the shoe with the dorsal border behind the tip of the frog you can cut out the toe of your original shoe and you will have a H-shoe. This shoe offers the positive effects of an open toe shoe without the need of having a bar at the palmar/plantar end of the shoe at the same time.

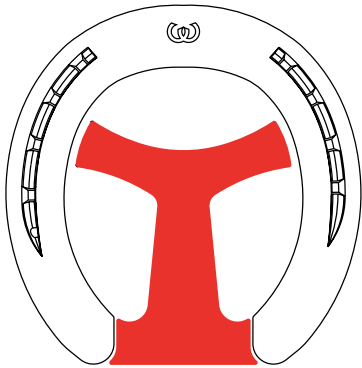SPIDER PLATE 2.0

FIELD OF APPLICATION

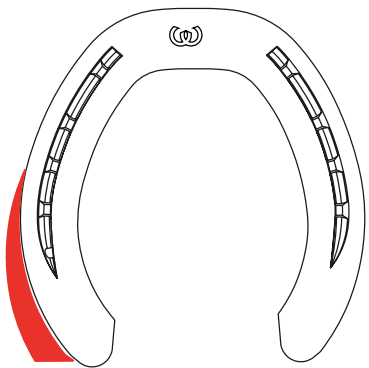
- Founder/Laminitis
- Pathologies of the deep flexor tendon
- Pathologies of the check ligament of the deep flexor tendon
- Chronic navicular disease
- Palmar foot pain
- Inflammations of the ligaments of the toe
- Hoof cracks

EFFECT

- The Spider plate gives you a large area of support or protection.
- By using it on different positions of the shoe you can determine which area of the shoe will penetrate yielding ground less or more.

LATERAL EXTENSION 2.0

FIELD OF APPLICATION

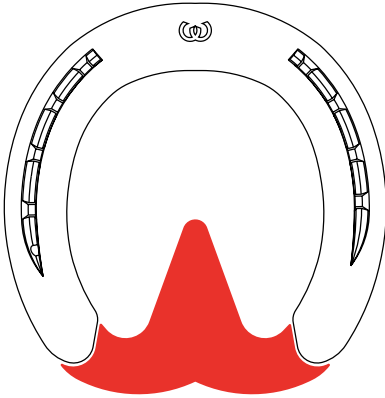
- Bone spavin
- Brushing

EFFECT

- The Lateral Extension can be used to give the horse extra support.
- It can widen the the horses gait and therefore help horses with “brushing” problems.
- Horses with bone spavin can as well benefit from lateral extension shoes.

HEART BAR 2.0

FIELD OF APPLICATION

- Founder/Laminitis
- Pathologies of the deep flexor tendon
- Pathologies of the check ligament of the deep flexor tendon
- Chronic navicular disease
- Palmar foot pain
- Inflammations of the ligaments of the toe
- Hoof cracks


EFFECT

- Expansion of the palmar support area without modification of the palmar lever arm.
- Reduces the risk of losing the horseshoe due to pulling the shoe (in comparison with shoeing with palmar extension).
- Increase of the angle of the coffin bone on soft soil.
- Reduction of the tension of the deep flexor tendon and a reduced load on the navicular area.
- Without frog contact: protection of frog and palmar hoof section (navicular bone, attachment of the deep flexor tendon) on all types of soil.
- With frog contact: continual increase of the load-bearing surface in the palmar hoof section, regardless of the type of soil.

DIMENSIONS 2.0

ORTHOKIT 2.0

Model	Size	mm	
		Height	Width
Z-Bar	Ortho Z-Bar L - 6 mm	80	95
	Ortho Z-Bar L - 8 mm	80	95
	Ortho Z-Bar M - 6 mm	66	85
Center Bar	Ortho Center-Bar L - 6 mm	37	112
	Ortho Center-Bar L - 8 mm	37	112
	Ortho Center-Bar M - 6 mm	32	96
Spider Plate	Ortho Spider Plate L - 4 mm	124	109
	Ortho Spider Plate M - 4 mm	97	93
	Ortho Spider Plate S - 4 mm	94	83
Lateral Extension	Ortho Lat. Ext. L - 8 mm	93	22
	Ortho Lat. Ext. M - 8 mm	81	18
	Ortho Lat. Ext. S - 8 mm	73	17
Heart Bar	Ortho Heart Bar L - 6mm	93	22
	Ortho Heart Bar M - 6 mm	81	18
	Ortho Heart Bar S - 6 mm	73	17

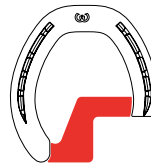Z- BAR SHOE

CENTER BAR SHOE

SPIDER BAR SHOE

LATERAL EXTENSION

HEART BAR SHOE

SIZES 2.0

SMALL

Size S fits flush for a Warrior front shoe size 0
(Equates to Ryder Original Size 2 / Euroskill size 0)

MEDIUM

Size M fits flush for a Warrior front shoe size 2
(Equates to Ryder Original Size 4 / Euroskill size 2)

LARGE

Size L fits flush for a Warrior front shoe size 5
(Equates to Ryder Original Size 7 / Euroskill size 5)

EXCEPTION “LATERAL EXTENSION BAR”

This bar fits flush for Warrior hind shoes in the corresponding sizes.
With the help of appropriate adjustment (grinding) these welding parts fit into different Werkman horseshoes.

COMMENTS 2.0

IMPORTANT

The ORTHOKIT modifications have different effects on different types of soil. It is important to consider the bedding in the box and type of soils that the horse is worked on for optimum effect.

Keep in mind that relieving the load on an anatomical structure will always be accompanied by an increase in the load in another area.

REFILL KITS

**Refill kits are available in all different models and sizes.
10 pieces of one size and one model per box.**

For more information regarding the use of Werkman ORTHOKIT, please contact: shop@werkmanhoofcare.com or visit werkmanhoofcare.com/academy

DISCLAIMER

The Werkman ORTHOKIT should only be selected, used, and mounted by farriers and veterinarians who possess adequate knowledge of orthopedic shoeing. Inexpert or erroneous selection or mounting of the Werkman ORTHOKIT (especially when welding the parts together) can undo the desired effect and even cause damage. Therefore, C.V. Werkman disclaims any liability for potential injuries, damage, costs or other consequences, caused by inexpert or erroneous selection and/or mounting of the Werkman ORTHOKIT.

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