




WERKMAN PADS

INFORMATION BOOKLET

CONTENTS

GENERAL	4	USE	13
		Align horseshoe on support pad	13
HOW	5	Using hoofsolution on support pad	14
		Align horseshoe on precision pad	15
WERKMAN LEATHER PADS	7	Using hoofsolution on precision pad	16
Dimensions leather pad (Z0014)			
Dimensions leather wedge pad			
WERKMAN SUPPORT PADS	9		
Sizes & Dimensions			
WERKMAN PRECISION PADS	11		
Sizes & Dimensions			

The Werkman Hoof Pads should only be used by farriers who possess adequate, professional knowledge. Inexpert or erroneous usage of the Werkman Hoof Pads may have an adverse effect and even result in damages. Werkman Hoofcare B.V. accepts no liability for potential injuries, damages, costs or other consequences, caused by the selection and/or use of the Werkman Hoof Pads

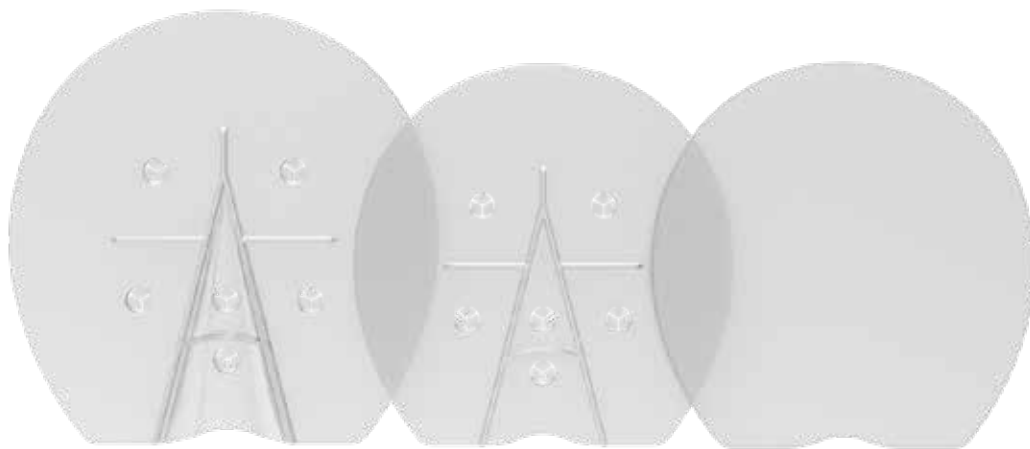

GENERAL

With the Werkman Pad Solutions we offer a total solution, consisting of high-quality leather and synthetic soles. This gives you a wide range with own specific characteristics, which offer the right solution for the horse.

With the introduction of these models, our aim is to provide comfort to the horse suffering from, for example, a weak hoof. In addition, we also have soles, which are used to cover, protect, elevate and support areas and structures on the ground side of the hoof. The soles can also be applied as part of a treatment in case of illness or damage.

In this brochure you will find more information about the contents and the corresponding specifications. If you would like to know more about our products, please contact your local Werkman Hoofcare dealer or our head office in the Netherlands via sales@werkmanhoofcare.com.

This booklet is intended to provide farriers with information about Werkman Hoof pad range. It does not give any set rules. Werkman Hoofcare therefore does not accept any liability for any injuries, damage, costs or other consequences of applying this information.




SMARTER SKILLS, BETTER HORSES

WERKMAN HOOF PADS

HORSE WELL-BEING

You know better than anyone that good horseshoes are vital to a horse's well-being.

At Werkman Hoofcare, we understand that you need top-quality horseshoes tailored to the anatomy of the hoof. The combination of our horseshoes and pads can take hoof vitality to the next level.

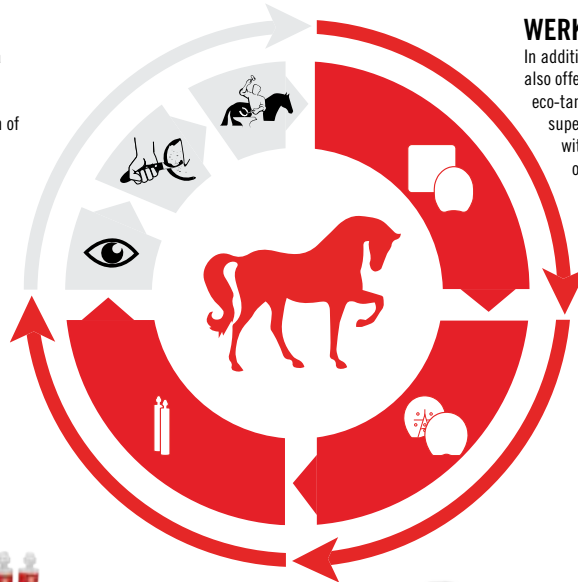

WERKMAN HOOF CARE SOLUTIONS

Use Werkman Hoofcare Solutions for the right sole protection, cushioning and support. The adhesives ensure an excellent bond between the various Werkman pads and are available in different hardness ratings.



WERKMAN LEATHER PADS

In addition to the classic Werkman leather pads, we now also offer Werkman wedge pads. These are made from eco-tanned breathable buffalo leather. Thanks to their superb bonding, they are excellent in combination with Werkman Hoofcare Solutions. The pads provide optimal protection that softens impacts.



COPPER RIVETS

Werkman copper rivets can be used to attach pads to the heels of the horseshoes. Thanks to their length (30 mm), they are usable with both 8mm and 10mm horseshoes.



WERKMAN PLASTIC PADS

With the new transparent plastic pads, you'll enjoy not only all of their special features, but also an excellent view of your work. The plastic pads provide effective cushioning and are highly resistant to wear. They can be stretched repeatedly without permanent deformation, while still retaining their structure and properties. Finally, Werkman plastic pads are highly resistant to oils and fats. In short, a perfect addition to your toolkit!



WERKMAN LEATHER PADS



SIZES & DIMENSIONS

Dimensions leather pad (Z0014)

Article code	Size	Model	Height [mm]	Width [mm]	Thickness [mm]
Z0014	M	Leather pad	160	160	3
Z0014A	L	Leather pad	180	180	3

Dimensions leather wedge pad

Article code	Size	Model	Height [mm]	Width [mm]	Thickness [mm]
WHS7001	M	Leather wedge pad	x	x	6 → 3
WHS7002	L	Leather wedge pad	180	155	6 → 3



WERKMAN SUPPORT PADS



FITTINGS

- Size M → Warrior Size 1 and 2
- Size L → Warrior Size 3 and 4

SIZES & DIMENSIONS

Article code	Size	Model	Height [mm]	Width [mm]	Thickness [mm]
M	M	Support pad	165	165	3
L	L	Support pad	190	190	3



WERKMAN PRECISION PADS



FITTINGS

- Size M → Warrior Size 1 and 2
- Size L → Warrior Size 3 and 4

SIZES & DIMENSIONS

Artikel code	Size	Model	Height [mm]	Width [mm]	Thickness [mm]
WHS6003	M	Precision pad (with frog support)	166	165	3
WHS6005	M	Precision pad	166	165	3
WHS6004	L	Precision pad	190	189	3
WHS6006	L	Precision pad (with frog support)	190	189	3



Werkman
Hooftare
oplossingen



Sole Cushion
Soft Shoe Adh

WERKMAN HOOF SOLUTIONS



Easy to use cartridges with screw cap to prevent the product from drying out.

Shock Absorbing Solution

Protection, cushioning and support of the sole.

Hoof Repair Solution

Repair, correction or extension of hooves.

WERKMAN HOOF CARE SOLUTIONS		
WHS1001	Werkman Sole Cushion Soft Shore A40	250 ml
WHS1002	Werkman Sole Cushion Medium Shore A50	250 ml
WHS2001	Werkman Hoof Fix White	250 ml
WHS2002	Werkman Hoof Glue&Fix Black	150 ml
WHS3001	Werkman Sole Support Transparant Shore 20	250 ml
WHS4001	Mixing Tips	
WHS4002	Universal Dispenser	
WHS4004	Adapter for 150 ml cartridges	



WERKMAN HOOF SOLUTIONS



WERKMAN HOOFCARE SOLUTIONS		
WHN1001	Copper rivets	30mm
HG0050	Kretzer leather scizzors	

